



# Refitting Monovision LENS WEARERS

Follow these best practices when fitting NaturalVue® (etafilcon A) Enhanced Multifocal 1 Day™ contact lenses—now with the TripleTear® Lubrication System and Ultra-Tapered Edge designed for optimal fit and comfort.

## Stepwise Approach to Refitting Long-term, Established, Well-adapted Monovision Contact Lens Wearers

Because monovision patients have adapted to the significant overplussing of their non-dominant eye, it can be challenging to refit them into any multifocal, especially a unique, extended depth of focus design like NaturalVue® Enhanced Multifocal 1 Day™ contact lenses.

The following approach has been found to minimize this issue:

### EXAMPLE:

The patient has been wearing -3.00D OD and -1.00D OS.

### SPECTACLE PRESCRIPTION:

OD: -2.75 -0.25 X 095

OS: -2.50 -0.75 X 090 ADD +2.00D OU

- The NaturalVue® Multifocal QuickStart Calculator recommends starting powers of -3.00 OU.
  - Dominant Eye: Start with -3.00D
  - Non-dominant eye: Start with -2.00D [(-3.00 suggested power + ½ the +2.00D ADD) therefore, (-3.00) + (+1.00) = -2.00 to start with on the non-dominant eye.]
- At the follow-up visit, you should be able to change the non-dominant eye power to -3.00D
- Generally, the vision will be excellent at Distance and Near at this point.
- If near still needs enhancement, add +0.25 D to the non-dominant eye.
- If further enhancements are needed, follow the Fitting Guide.

For product enquiries or fitting support, please contact Soe Wong, Professional Service Specialist, Oculus Optical (Hong Kong), at [soe.wong@oculuslens.com](mailto:soe.wong@oculuslens.com) or 852 98426161.